

**Tailback42** 



**MANAMED**™

Patents Pending.

# Tailback42™




The Tailback Brace provides support for both spinal surgeries and conservative treatment of spinal injuries. The brace features universal sizing, an ice pack for optimal patient experience, and multiple L code configurations. The entire Tailback spine line features our patent pending chord compression system and signature M shaped handle.

## Sizing

Tailback can be adjusted from 25" to 68". Tailback comes with a main panel and two adjustable belts with comfort pull rings. Simply place the fullback on a table with the inside facing up. When using each matching number, the waist is measured as the following:

S & S = 25" - 32" waist	2XL & 2XL = 50" - 56" waist
M & M = 32" - 38" waist	3XL & 3XL = 56" - 62" waist
L & L = 38" - 44" waist	4XL & 4XL = 62" - 68" waist
XL & XL = 44" - 50" waist	

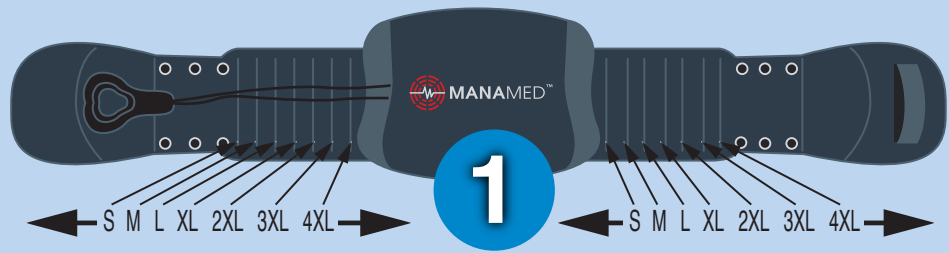

**Step 1** Match the size using the chart to fit your waist match your belts size. Remove Velcro from belts and pull the tabs through to the sizing creases, securing the sizing flap with Velcro to your appropriate crease 1,2,3,4,5,6,7. For your appropriate size, see chart above and reference **Sizing** box to the left (in blue).

## Features and Benefits


- Universal size with Low-profile design
- Simple One Pull Compression System
- Ice Pack
- Three Location Gear System
- Multiple L Code Configurations

## Indications for Use

- |                       |                   |
|-----------------------|-------------------|
| Low Back Pain         | Laminectomy       |
| Osteoporosis          | Discectomy        |
| Disc Degeneration     | Spinal Stenosis   |
| Disc Herniation       | Spondylolisthesis |
| Post-Operative Fusion |                   |

## Ice Pack


Fits easily in the brace, attaches with Velcro.

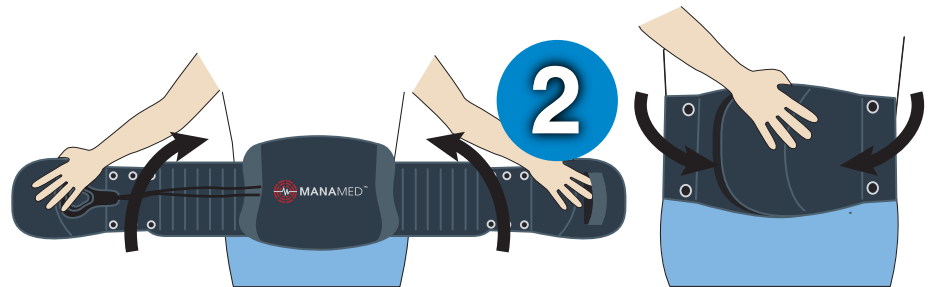

**Step 2** Put the brace around your waist and secure the front panel onto the brace so the Velcro secures the fit.



**Step 3** Pull both comfort rings forward until you feel comfortable and then secure the rings to your brace by placing them on to the sides. The comfort rings control the compression of the brace and can be used to adjust the fit.

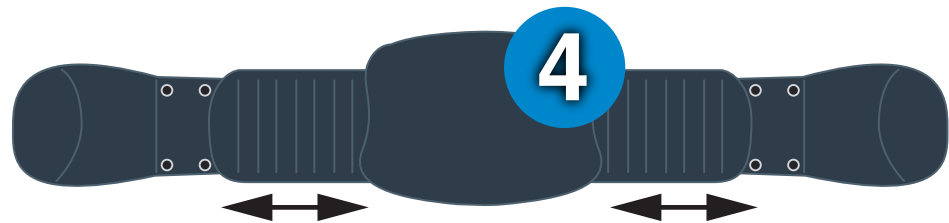

**Removing the brace** Lift both comfort rings off the brace placing them to the sides and securing them so there is no tension on the brace. Then remove the Velcro connection on the front panel and safely remove the brace.

## Order information

Part Number: TB042      Size: Universal

## How to order

Email: [orders@manamed.net](mailto:orders@manamed.net), Fax: 888-508-0712, Call: 888-508-0712

1511 W. Alton Ave.,  
Santa Ana CA 92704  
[www.ManaMed.Net](http://www.ManaMed.Net)



MANAMED™