



 **MINI
KAHUNA™**



MANAMED™

Patents Pending.



The Mini Kahuna Brace is universal and can be applied to right or left side. The breathable material allows for comfort and optimal patient experience. The Mini Kahuna brace fits the description of the L3670 HCPC. Mini Kahuna comes with abduction pillow and ball.


Intended Use

- Arthroscopic Repairs
- Anterior Repairs
- Rotator cuff repairs
- Shoulder instabilities



Application of the Kahuna Brace:


1. Open Velcro strap #1 and #2, also open buckle on the arm sling (buckle #3). Place arm in the brace and secure both Velcro strap #1 & #2 comfortably around arm. To fit your arm into the sling, you can fold the sling along the pre-folded point under the arm sling buckle #3.

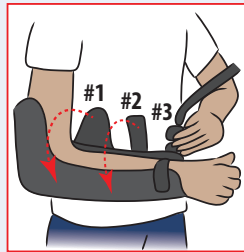

2. Place shoulder belt #4 around the opposite shoulder and adjust the sling #5 to fit comfortably around the shoulder.




3. Position the back support pad #6 in the middle of the patients back. Then adjust the torso strap by adjusting the belt loop #7 which is at the base of the buckle #8.

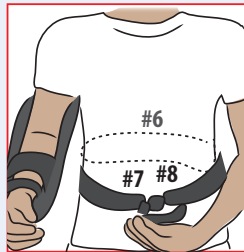

4. Position the abduction pillow #6 on the injured side of the body.



5. Make final adjustments for comfort. Adjust periodically if needed.

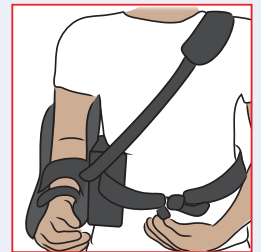

Washing and Care:

The Mini Kahuna brace may be cleaned by hand using mild soap and warm water. Hang to dry. Do not place in washing machine or dryer.

Warning: The Mini Kahuna brace is supportive only and does not guarantee or intend to prevent injury. Consumer or patient agrees to the following.

- ManaMed accepts no responsibility for any injury which might occur while using ManaMed products.
- Skin irritation may occur while using this device. If you experience an unusual skin reaction while using this product, consult with your healthcare provider immediately.
- Consult your health care provider for this products indication for use.
- For single patient use only.
- Federal Law restricts this device to sale or by or on the order of a licensed health care practitioner.
- If you do not agree to the above conditions, contact your health care provider before using this product.

Order information

Part Number: MKB001 Size: Universal

How to order

Email: orders@manamed.net, Fax: 888-508-0712, Call: 888-508-0712

1511 W. Alton Ave.,
Santa Ana CA 92704
www.ManaMed.Net



MANAMED™